

# Baby.i Wristband

## User Manual

---




## Getting Started

Thank you for choosing the [Baby.i Wristband](#) connected with Bluetooth. [Baby.i Wristband](#) is a kind of Bluetooth Low-energy product based on the Bluetooth 4.0 version. It can be wearable on baby's hands & legs, and linked up smartphones through specified app within a setting Bluetooth connection. [Baby.i Wristband](#) can measure and record the baby's body temperature and movement data, parents can monitor those data through smartphone after connection. [Baby.i Wristband](#) app provides the notifications of feeding, diaper change and body temperature alert.

With our low-energy technology for Baby.i Wristband, CR2032 Lithium Coin Battery life can be extended.

### Included:

- Main Device
- Silicone belt
- CR2032 battery x 1
- User manual x 1 sheet

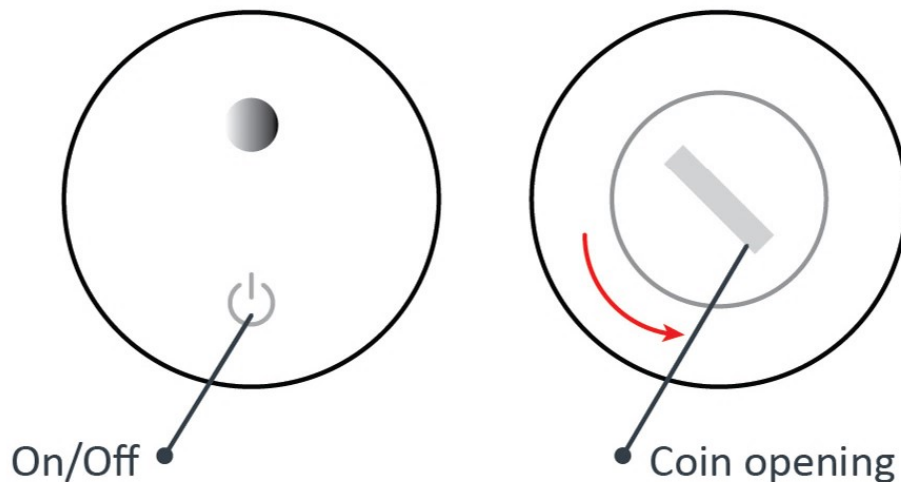

## Switch on/off

Switch on: Press and hold the on/off button for 2 seconds until you see the light is on, and then flash light continuously on.

Switch off: Press and hold the on/off button for 2 seconds until the short flash light is on.

## Connection

1. Switch on **Baby.i Wristband** and turn on your smartphone Bluetooth.

2. Open the **Baby.i Wristband** app.



3. Select device to connect on device connection page. For 1st time connection, select the name setting page to create the baby name and favorite icon. (You can rename later when you click the icon button in control page.)


4. Click the button to connect the device.




## Multi devices connection

1. **Baby.i Wristband** app allows the smartphone to connect more than 1 device.


2. On connection page, click the icon to switch the device.

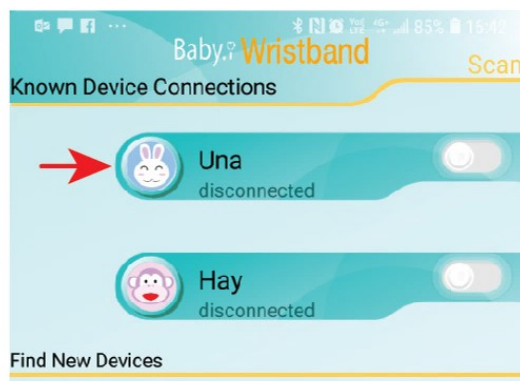


## Baby.i Wristband App Operation

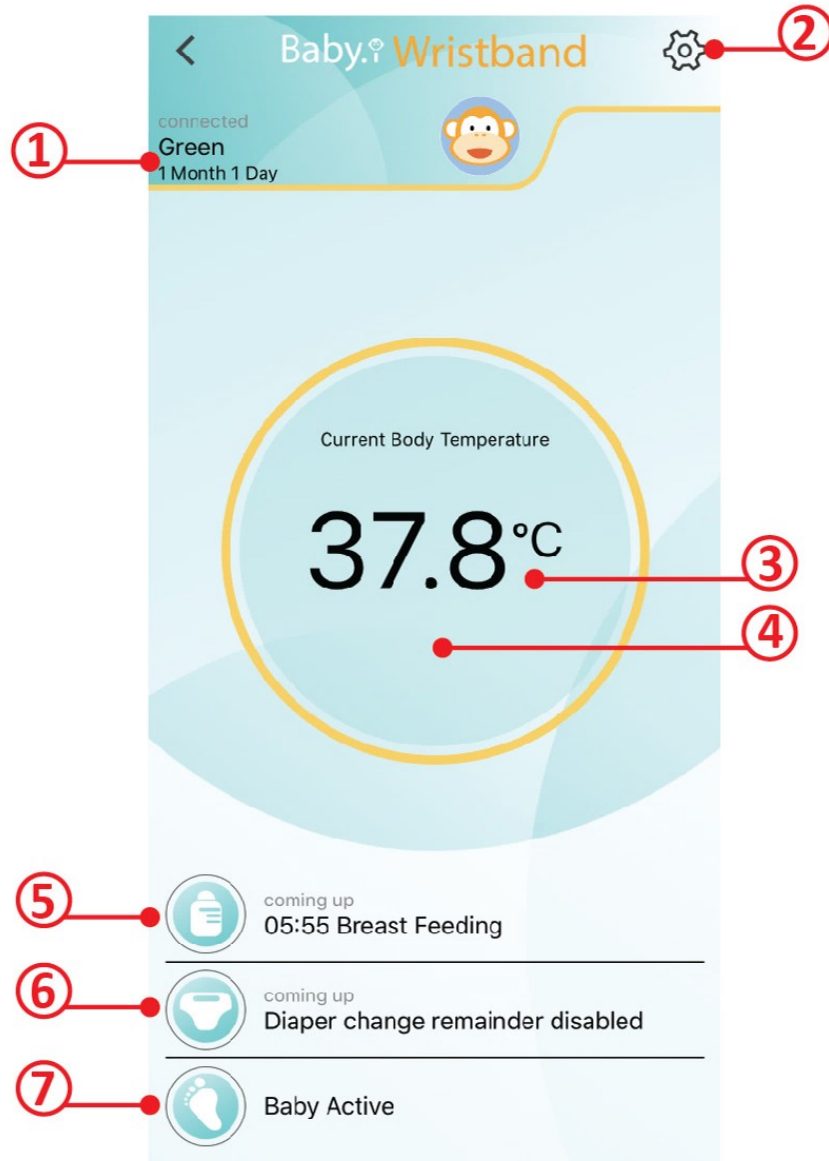

**① Baby details:** Rename, date of birth, gender and icon setting.

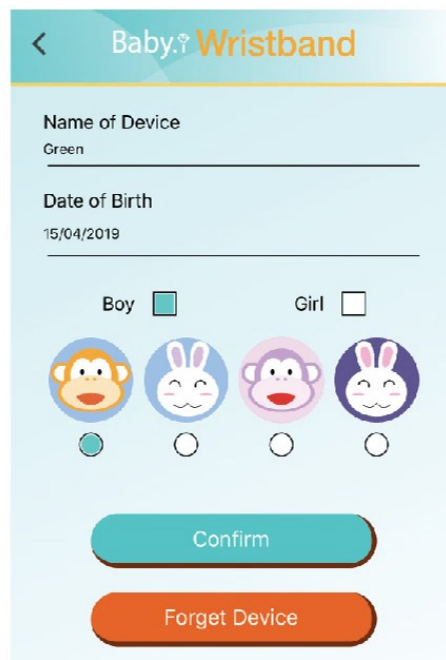


**② Setting:**  
- Body temperature alert

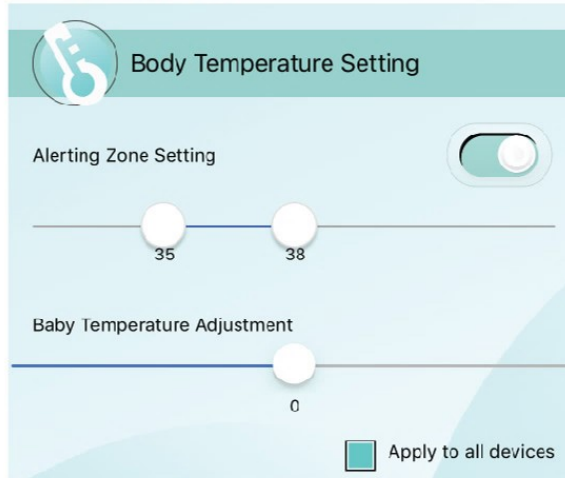


The 'Body Temperature Setting' screen features a teal header with a thermometer icon and the title. Below the header, there is a toggle switch for 'Alerting Zone Setting' which is currently turned on. Underneath the toggle is a horizontal slider with two white circular handles; the left handle is positioned at '35' and the right handle is at '38'. Below the slider is another horizontal slider for 'Baby Temperature Adjustment' with a single white circular handle positioned at '0'. At the bottom right, there is a checkbox labeled 'Apply to all devices' which is currently checked.

- Feeding (you decide how many and what kind of meals, then add the time of 1st meal. Feeding schedule are listing out and notificate you on time.)

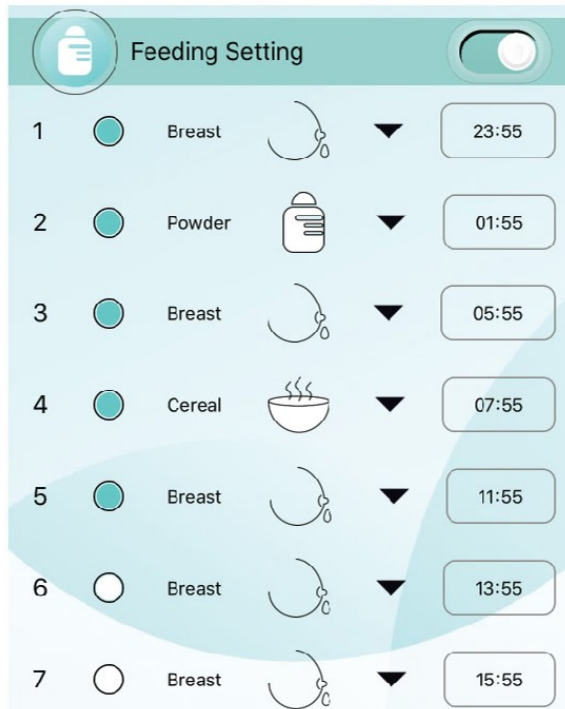

The 'Feeding Setting' screen has a teal header with a baby bottle icon and the title. A toggle switch for the setting is turned on in the top right corner. The main area contains a list of seven feeding entries, each with a number, a selection circle, a food type, an icon, a dropdown arrow, and a time field. The entries are: 1. Breast (selected), 2. Powder (selected), 3. Breast (selected), 4. Cereal (selected), 5. Breast (selected), 6. Breast (not selected), and 7. Breast (not selected). The times for entries 1 through 5 are 23:55, 01:55, 05:55, 07:55, and 11:55 respectively. Entries 6 and 7 have times of 13:55 and 15:55 respectively.


- Change diaper (you decide how long before / after the meal. Notification for you on time.)

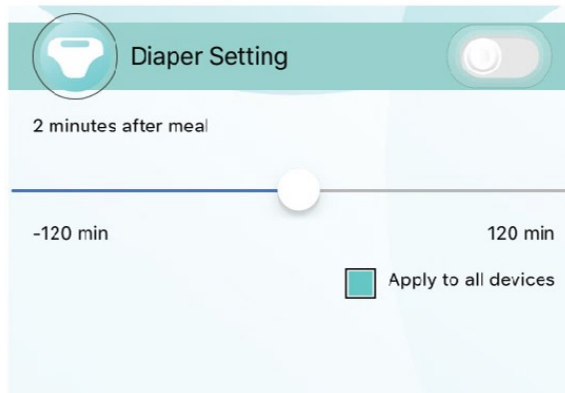

The 'Diaper Setting' screen features a teal header with a diaper icon and the title. A toggle switch for the setting is turned on in the top right corner. Below the header, the text '2 minutes after meal' is displayed. Underneath is a horizontal slider with a white circular handle positioned exactly in the middle. The left end of the slider is labeled '-120 min' and the right end is labeled '120 min'. At the bottom right, there is a checkbox labeled 'Apply to all devices' which is currently checked.



- ③ Current body temperature
- ④ Body temperature chart

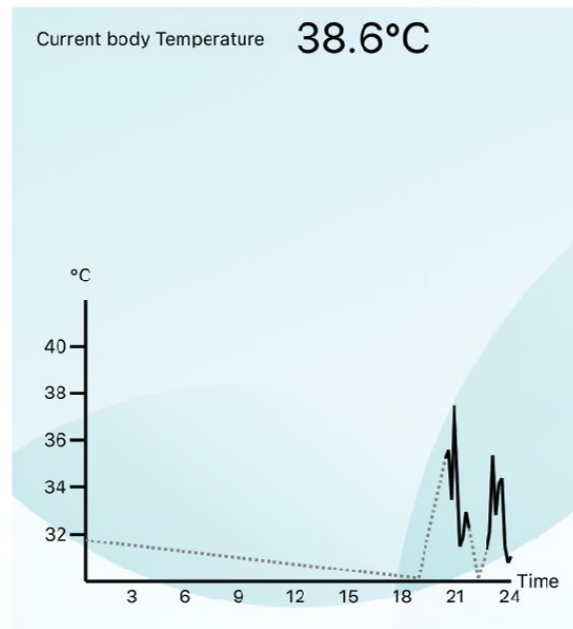


- ⑤ Feeding schedule
- ⑥ Change diaper schedule
- ⑦ Body active - Record and analysis the amount of baby movement per day.

Attention: If you run the Baby.i Wristband app in backend operation, ie launch another apps or lock the device, the app may not be able to wake up and display the ( temperature / cry ) alert. This is due to iOS protection against draining system resources or Android authority limitation. We strongly recommend to keep the app in active state when monitoring and alert function is necessary.

### Baby.i Wristband App

App can be downloaded from;

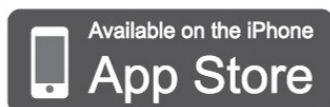

Thank You

[www.innohub-iproduction.com](http://www.innohub-iproduction.com)

